Quantitative determination of Apolipoproteins (APO)
IVD

Store at 2 - 8°C.

PRODUCT CHARACTERISTICS

APO CAL is a lyophilised multicalibrator serum, from human origin, containing apolipoprotein concentrations suitable to calibrate determinations of apolipoproteins by turbidimetry method.

REAGENTS

APO-CAL	Calibrator. Human serum. Preservative. APO concentrations are stated on the list below.
---------	--

PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

H317: May cause an allergic skin reaction.

Follow the precautionary statements given in MSDS and label of the product.

CALIBRATION

The concentration values of the Calibrator are traceable to Secondary Reference Material Apo A-I WHO / IFCC (SP1-01) and Apo B WHO / IFCC (SP3-07), and Internal References Materials for APO C-II, C-III and E.

PREPARATION

Reconstitute (→) with 1 mL of distilled water. Mix thoroughly, avoiding foam forming.

The resulting calibrator solution should be used immediately or stored at 2-8°C.

STORAGE AND STABILITY

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

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