HbA<sub>1c</sub>

# Glycated Hemoglobin

Latex turbidimetry

## Quantitative determination of glycated hemoglobin (HbA<sub>1c</sub>) in human blood

### IVD

Store 2 - 8°C.

### PRINCIPLE OF THE METHOD

This method utilizes the interaction of antigen and antibody to directly determine the HbA<sub>1c</sub> in whole blood. Total hemoglobin and HbA<sub>1c</sub> have the same unspecific absorption rate to latex particles. When mouse antihuman HbA<sub>1c</sub> monoclonal antibody is added (R2), latex-HbA<sub>1c</sub>-mouse anti human HbA<sub>1c</sub> antibody complex is formed. Agglutination is formed when goat anti-mouse IgG polyclonal antibody interacts with the monoclonal antibody. The amount of agglutination is proportional to the amount of HbA<sub>1c</sub> absorbed on to the surface of latex particles. The amount of agglutination is measured as absorbance. The HbA<sub>1c</sub> value is obtained from a calibration curve.

### CLINICAL SIGNIFICANCE

Throughout the circulatory life of the red cell, Hemoglobin A1c is formed continuously by the adduction of glucose to the N-terminal of the hemoglobin beta chain. This process, which is non-enzymatic, reflects the average exposure of hemoglobin to glucose over an extended period. In a classical study, Trivelli et al<sup>1</sup> showed Hemoglobin A1c in diabetic subjects to be elevated 2-3 fold over the levels found in normal individuals. Several investigators have recommended that Hemoglobin A1c serve as an indicator of metabolic control of the diabetic, since Hemoglobin A1c levels approach normal values for diabetics in metabolic control.<sup>2,3,4</sup> Hemoglobin A1c has been defined operationally as the "fast fraction" hemoglobins (HbA<sub>1a</sub>, A<sub>1b</sub>, A<sub>1c</sub>) that elute first during column chromatography with cation-exchange resins. The non-glycosylated hemoglobin, which consists of the bulk of the hemoglobin has been designated HbA<sub>0</sub>. The present procedure utilizes a antigen and antibody reaction to directly determine the concentration of the HbA<sub>1c</sub>.

### REAGENTS

R1	Latex 0,13%, Buffer, stabilizer.
R2	Mouse anti-human HbA <sub>1c</sub> monoclonal antibody 0,05mg/mL, goat anti-mouse IgG polyclonal antibody 0,08mg/dL, Buffer, stabilizers.
R3 (Hemolysis reagent)	Water and stabilizers

### PRECAUTIONS

All human specimens should be regarded as potentially biohazardous. Therefore, universal precautions should be used in specimen handling (gloves, lab garments, avoid aerosol production, etc.).

### PREPARATION

R1, R2 and R3 are ready to use. Mix gently before use.

### CALIBRATION

Recalibrate when control results are out of specified tolerances, when using different lot of reagent and when the instrument is adjusted.

### STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Reagents should not be left inside the analyzer after use, they must be stored refrigerated at 2-8°C. Latex may sediment. Mix reagents gently before use. Do not use reagents over the expiration date. R1 and R2 are stable for at least one month after opening stored at 2-8°C.

Hemoglobin A1c in whole blood collected with EDTA is stable for one week at 2-8°C.<sup>5</sup>

**Reagent deterioration:** Alterations in the physical appearance of the reagents or values of control materials outside of the manufacturer's acceptable range may be an indication of reagent instability.

### ADDITIONAL EQUIPMENT

- SPIN 800 autoanalyzer.
- Laboratory equipment.

### SAMPLES

Special preparation of the patient is unnecessary. Fasting specimens are not required. No special additives or preservatives other than anticoagulants are required. Collect venous blood with EDTA using aseptic technique.

To determine HbA<sub>1c</sub>, a hemolysate must be prepared for each sample:

1. Dispense 1 mL Hemolysis Reagent into tubes labeled: Calibrator, Control, Patients, etc. Note: Plastic or glass tubes of appropriate size are acceptable.
2. Place 20 µL of well mixed whole blood into the appropriately labeled lyse reagent tube. Mix.
3. Allow to stand for 5 minutes or until complete lysis is evident. Hemolysates may be stored up to 10 days at 2-8°C.

### REFERENCE VALUES

Recommended Values: less than 6% for a non-diabetic, less than 7% for glycemic control of a person with diabetes.

Each laboratory should establish its own expected values. In using Hemoglobin A1c to monitor diabetic patients, results should be interpreted individually. That is, the patient should be monitored against him or herself. There is a 3-4 week time lag before Hemoglobin A1c reflects changes in blood glucose level.

### QUALITY CONTROL

HbA<sub>1c</sub> Control (ref: 43106) is recommended to monitor the performance of manual and automated assay procedures. **Controls require hemolysis pretreatment after being reconstituted.**

Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

**BARCODED REAGENTS LOAD MUST BE PRECEDED OF A SPINREACT "DATABASE" COPY INTO THE ANALYZER SOFTWARE. IT IS AVAILABLE UNDER REQUEST TO SPINREACT.**

### SPIN 800 APPLICATION

EDIT PARAMETERS			
Test	HbA <sub>1c</sub>	No.	**
Full name	HbA <sub>1c</sub>	Print name	HbA <sub>1c</sub>
Reac. Type	End Point	Direction	Increase
Pri. Wave.	660	Sec. Wave.	
Unit	%	Decimal	0,01
Reagent Blank	-	React. Time	32 - 33
Vol. Sample	5 ul	Increase	Decrease
R1	180 ul	R2	60 ul
		R3	R4
		R4	
CALIBRATION			
Spline	Water	0	
	Calibrator	Cal Level 1	
	Calibrator	Cal Level 2	
	Calibrator	Cal Level 3	
	Calibrator	Cal Level 4	

\*\* Modify reagents and sample volumes according to the range accepted but keeping always the mentioned ratio.

### PERFORMANCE CHARACTERISTICS

**Measuring range:** From detection limit of 2% to linearity limit of 16%.

#### Precision:

	Intra-assay (n=20)		Inter-assay (n=20)	
	Mean (%)	SD	CV (%)	
Mean (%)	5,95	12,15	5,97	12,21
SD	0,19	0,18	0,14	0,15
CV (%)	3,20	1,47	2,31	1,24

**Sensitivity:** 1% = 0,056 (A)

**Accuracy:** Results obtained using SPINREACT reagents (y) did not show differences when compared with other commercial reagent (x). The results obtained using 40 samples were the following:

Correlation coefficient (r)<sup>2</sup> 0,995

Regression equation: y = 0,989x - 0,047

The results of the performance characteristics depend on the analyzer used.

### LIMITATIONS

1. This assay should not be used as the unique test for the diagnosis of diabetes mellitus. Other tests must be also considered to establish a correct diagnosis.
2. Patient specimens should always be assayed using a calibration curve.
3. It has been reported that results may be inconsistent in patients who have the following conditions: opiate addiction, lead-poisoning, alcoholism, ingest large doses of aspirin.<sup>6, 7, 8, 9</sup>
4. It has been reported that elevated levels of HbF may lead to underestimation of HA1c and, that uremia does not interfere with HbA<sub>1c</sub> determination by immunoassay.<sup>10</sup> It has been reported that labile intermediates (Schiff base) are not detected and therefore, do not interfere with HbA<sub>1c</sub> determination by immunoassay.<sup>5</sup>
5. It has been determined that Hemoglobin variants HbA<sub>2</sub>, HbC and HbS do not interfere with this method.
6. Other very rare variants of hemoglobin (e.g. HbE) have not been assessed.

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